Askina® PICC Universal

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INSTRUCTIONS FOR USE

1. PRODUCT DESCRIPTION
Askina® PICC Universal is a sterile catheter stabilization device which can hold and secure a variety of catheter tube sizes and shapes. The device has a uniquely shaped foam insert which conforms to the shape of the catheter hub and locks the tube securely in place and a hook and loop system so that the tubing can be easily positioned and adjusted as needed.

2. INDICATIONS
Askina® PICC Universal is indicated for short to long term securement of most types of PICC tubing. It is indicated for external use only, and is placed under medical supervision.

3. PRECAUTIONS
Discontinue the use of this device if an allergic reaction occurs. Do not resterilize Askina® PICC Universal.

4. CONTRAINDICATIONS
Caution if known allergy to adhesives.

5. INFORMATION FOR USE
Askina® PICC Universal is very simple to apply and requires no special skills or equipment; it is intended for single patient use.

5.1 Skin Preparation
Note: The PICC catheter should already be inserted.

Select the area to place the Askina® PICC Universal device. Clean and dry the skin according to standard, local protocols. Hair removal may be required on some patients for better device fixation.

5.2 Dressing Application

Step 1:
Remove device from packaging, expose the foam channel by pulling the blue tab and slide the device under the catheter hub as above.

Step 2:
Position the catheter hub into the foam channel and press gently to the adhesive. Gently remove one of the folded white release tabs and apply the device on the skin. Gently remove the second folded release tab and smooth the device into place.

Step 3:
Remove the release tab from the top flap to expose the adhesive band.

Step 4:
Close the top flap of the catheter hub. Gently press to ensure that it is closed securely. The top flap is easy to open to inspect the position of the catheter and to adjust it if needed.

5.3 Removal of Dressing
(i) Gently open the top flap by pulling the blue tab
(ii) Remove the tubing from the foam channel in the device
(iii) An alcohol swab will ease device removal from the skin
(iv) Follow procedure 5.2 to apply a new Askina® PICC Universal device

6. SPECIAL NOTES
Askina® PICC Universal should be changed every 7 days, or changed according to the local protocol. Prolonged exposure to water may affect the adhesion of Askina® PICC Universal to the skin, reducing wear time.

7. WARNING
Do not use if pack is damaged or opened.

8. PRESENTATION
Askina® PICC Universal is available in one size: 88 x 37mm

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