

Askina® PICC Universal

Catheter securement device



Simple & Safe. Universal.

Askina® PICC Universal

Catheter securement device

Askina® PICC Universal is a sterile catheter stabilization device which can hold and secure a variety of catheter tube sizes and shapes. It has a uniquely shaped foam insert which conforms to the shape of various catheter hubs and locks the tube securely in place. The hook and loop system is easy to position and adjust as needed.

Hook and loop flap:
easy to peel, to allow inspection
and adjustment of the catheter
hub

Gentle inert skin contact adhesive:
soft and secure adhesion,
not irritating to the skin

Foam insert adapted to
the shape of catheter
hub for proper position-
ing and safe hold

For patient's comfort and safety

S o g e n t l e

INDICATION

Short to long term securement of all types of PICC tubing

BENEFITS

- Breathable, soft and flexible material
- Gentle adhesive: hypoallergenic and latex-free, not irritating to the skin
- Reduced risk of accidental pull-off by use of adapted foam insert

SAFE FOR USE

- Lower risk of complication compared to other methods of fixation*:
- Better fixation than tapes: lower risk of dislocation and migration of the catheter
- Lower risk of infection and less traumatic compared to fixation by sutures

* Infusion Nursing Standard of Practice for catheter stabilization J. Infus. Nursing 2006; 29(IS):S1-S92

1

Before application clean and dry the skin according to the standard protocol. Expose the foam channel by pulling the blue tab and slide the device under the catheter hub. Position the catheter hub into the foam channel and press gently to the adhesive.



2

Gently remove one of the folded white release tabs and apply the device on the skin. Gently remove the second folded release tab and smooth the device into place.



3

Remove the release tab from the top flap to expose the adhesive band.



4

Close the top flap of the catheter hub. Gently press to ensure that it is closed securely. The top flap is easy to open to inspect the position of the catheter and to adjust it if needed.



Easy to apply, inspect and adjust

S a f e f o r u s e

UNIVERSAL

The shape of foam insert is adaptable to all kind of catheter connections:



ASKINA® PICC UNIVERSAL

Catheter securement device

Size 88 x 37 mm

Pack 50 pieces

Art. no. 7779450



