en Instructions for use

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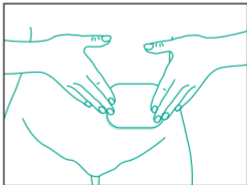
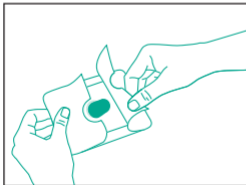
da Brugsanvisning

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sv Bruksanvisning

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en Instructions for use **Iryfix®**

The Iryfix® stoma cap is indicated for patients with a left colostomy who irrigate and whose stoma does not exceed 35 mm.

It should be used right after the irrigation for occasions where collecting effluents is not necessary. It allows gas adsorption without noise. It provides discretion for a few hours. Beyond that limit, it is recommended to wear a pouch specifically designed for output collection.

Patient population: Iryfix® is to be used on adults.

Special note: Children, pregnancy and breastfeeding women: Studies realized in animal models are insufficient to conclude on the toxicity of Iryfix® components over children, pregnant or breastfeeding women. There is no available data on the use of Iryfix® in these specific populations. In the absence of data, it is not advised to use Iryfix® in these cases.

Skin preparation: Wash your hands. Clean the skin around your stoma using only water. Dry the skin carefully. Avoid ether, alcohol or any antiseptic which could interfere with the skin protector adhesion.

Applying Iryfix®: Peel off the protective paper from the adhesive. Place Iryfix® in position while gently introducing the filter into the stoma. Smooth the adhesive onto the skin ensuring that there are no wrinkles. Apply light pressure to ensure complete adhesion.

Removing Iryfix®: To remove Iryfix® unstick one corner of the adhesive and peel off gently.

National regulations for the destruction of potentially infectious materials must be observed.

Warnings: Re-use of single-use devices creates a potential risk to the patient. It may lead to contamination and/or impairment of functional capability. In the rare event of sensitivity or allergic reaction to Iryfix® or any of its materials, discontinue use of the medical device. If your stoma morphology is not adapted, or if the diameter of your stoma exceeds 35mm, Iryfix® will not fit correctly. It is not advised to use Iryfix® in these cases. Colostomy irrigation itself is contra-indicated under the following circumstances: Current chemotherapy treatment, Induced radiation damages to the bowel, Irritable bowel syndrome, Crohn's disease, Diverticulitis, Peristomal hernias, Diarrhoea, Ulcerous inflammation of the large intestine, Improper fistula.

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FR B. BRAUN MEDICAL SAS
GB B. Braun Medical Ltd.
IT B. Braun Milano S.p.A
NL B. Braun Medical B.V
AT B. Braun Austria GmbH
CH B. Braun Medical AG